



- 1 - Cellular Connection "signal bar"
- 2 - Transmitting
- 3 - Network Connection "home light"



# GETTING STARTED

# Questions?

CALL 1-877-219-6290

1




Plug the hub into a power outlet near (within 10 feet) your testing meters.

2

The signal bar and home icons light up to indicate activity. This process may take up to 5 minutes to complete. When finished both the signal bar and the home light will remain solid green and yellow.

When ready, the hub will automatically search for your medical device. The signal bar and the home light will change to solid green when successfully paired with your device.

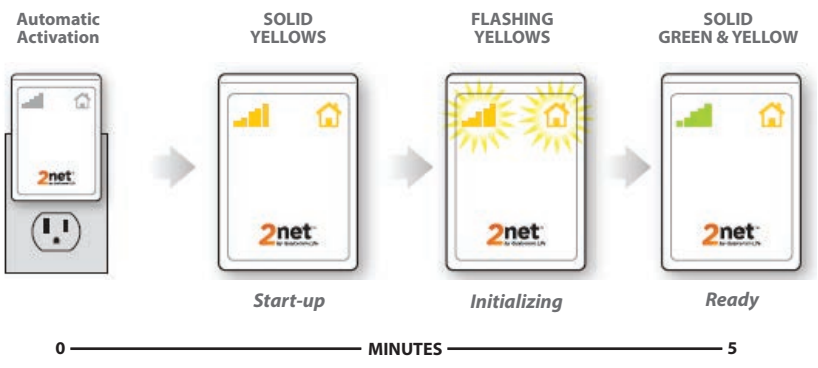



1

## Take your measurement



After testing press the UP button on your Meter. The LCD display on the Meter will show time counting up in seconds. The test results from your meter will be sent to the hub.



Both the weight scale and the blood pressure cuff will automatically transmit the results after complete your test. The hub will then transmit your test results.

2



The home light will flash green while it receives data from the meter.

3



The signal bars will begin to flash while the hub is connecting to the network. A blue light on top of the device will light up when the transmission has been successful.