




YOUR BLOOD PRESSURE MONITOR



- 1 - Air Socket
- 2 - Display
- 3 - Start Button
- 4 - Index Mark
- 5 - Arm Cuff
- 6 - Air Hose




MONITOR DISPLAY

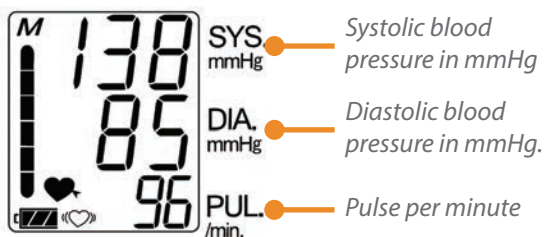


Questions?

CALL 1-877-219-6290

Standby and power on.

 The indicator that appears while measurement is in progress. It blinks when the pulse is detected.

 Irregular Heartbeat indicator. (I.H.B.) The indicator that appears when an irregular heartbeat or any excessive body movement is detected during the measurement.

 Full Battery

 Low Battery

Err *Unstable blood pressure due to movement during the measurement.*

The systolic and diastolic values are within 10 mmHg of each other.

The pressure value did not increase during inflation.

Err
CUFF
Err

The cuff is not fastened correctly.

PUL DISPLAY ERROR

The pulse is not detected correctly.

GETTING STARTED



1



Open the battery cover

2



Insert "AA" batteries and insert battery cover

3



Connect the “L” shaped air connector to the air hose

4



Insert the air connector plug into the air socket firmly



TAKING YOUR BLOOD PRESSURE

1



Wrap the cuff around the upper arm, about 2 - 3 cm above the elbow, as shown. Place the cuff directly against the skin, as clothing may cause a faint pulse, and result in a measurement error.

2



Constriction of the upper arm, caused by rolling up a shirtsleeve, may prevent accurate readings.

Confirm that the index points within the proper fit range.